


How to assemble and use the Sharpsafe® 2 & 3 Litre sharps container

ASSEMBLY INSTRUCTIONS

- 1 Place base of container on a firm surface for assembly (e.g. a table).
- 2 Attach lid to base of container by firmly applying pressure to each corner in turn (Fig.1).
- 3 Ensure all four corners engage with a “click”.
- 4 Complete relevant sections of label including name and date of assembly.

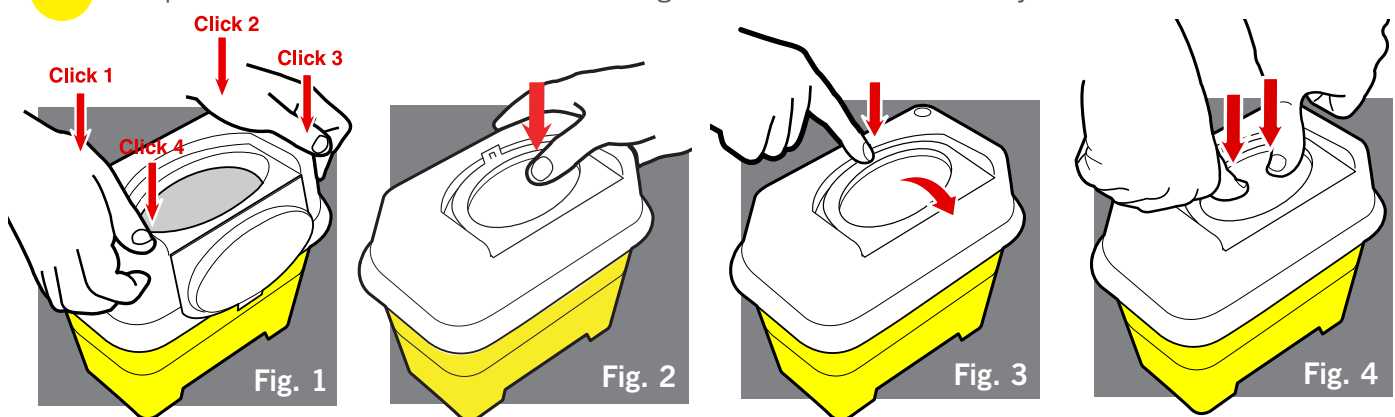

USER INSTRUCTIONS

- 1 Temporary closure:- Rotate cover over the aperture. Apply light pressure until a “click” is heard (Fig.2).
- 2 Temporary closure release:- Push tab (Fig.3). Cover will release and open.
- 3 Do not fill above line indicated where clear lid meets yellow base.
- 4 Final closure:- Rotate cover over the aperture applying firm pressure with thumbs to indentations (Fig.4). Ensure tab and both side clips engage.
- 5
- 6 Complete all relevant sections of the label including name and date of closure.

